

CLEAR + BRILLIANT[®] TOUCH

Treatment Settings and Pearls



Clear + Brilliant[®] Touch

The Clear + Brilliant[®] Touch System is a non-ablative laser designed for use in dermatological procedures for patients who want to manage the visible signs of aging skin. The system is part of the family of Solta Medical products utilising the Fractional Photothermolysis principle.

TREATMENT INFORMATION

The system includes the **Original Handpiece (1440 nm)** and the **Permêa[®] Handpiece (1927 nm)**, allowing for targeted treatment of different tissue depths.

Treatment Tips are single-use and interchangeable between handpieces during a session.

A **Credit Key** is required to activate treatments and is loaded with 12 treatment credits. One treatment credit is limited to a 3200 cm² of treatable surface area or a 40-minute window, whichever comes first.

TREATMENT PEARLS

PATIENT EXPECTATIONS	<ul style="list-style-type: none"> Discuss and set proper patient expectations Share before and after photos
TREATMENT PREPARATION	<ul style="list-style-type: none"> Ensure Credit Key has treatment credits available and has been successfully authenticated. Thoroughly cleanse treatment area(s), apply topical anesthetic as directed. Completely remove topical anesthetic by first wiping skin with dry gauze followed by wiping with alcohol or witch hazel. Allow skin to dry completely before initiating treatment. Apply appropriate patient protective eyewear. Ensure operator laser safety goggles are rated OD 5+.
SYSTEM CONSIDERATIONS	<ul style="list-style-type: none"> Utilise the Pass Counter and Charting Assistant features on Clear + Brilliant[®] Touch. Export treatment data to USB to keep record of patient's treatment.
SAFETY CONSIDERATIONS	<ul style="list-style-type: none"> The diode laser is contained in the handpiece. Exercise caution when handling the handpiece to minimise risk of damage. Place handpiece in cradle when not in use by operator. Do not transport the system with the handpieces attached and/or in the cradle. The Clear + Brilliant[®] Touch laser targets the water content in the skin and therefore does not selectively interact with melanin content. A patient's skin type and its melanin content, however, have an impact on the manifestation of skin reactions to laser injury. Adjust the treatment level accordingly for skin of color and sensitive skin types to reduce the risk of unwanted side effects such as post-inflammatory hyperpigmentation (PIH).

RECOMMENDED CLEAR + BRILLANT® COMPLETE TREATMENT PROTOCOL

Complete steps 1-10 with the Original Handpiece first.
Next, switch to the *Perméa*® Handpiece and repeat steps 1-10.

1 Start at edge of treatment area and perform one vertical pass

2 Perform a second vertical pass over the same line

3 Move over 100% (no overlap) and perform one vertical pass

4 Perform a second vertical pass over that same line

5 Repeat process until entire cosmetic zone is treated — these are passes one and two

6 Start at edge of treatment area and perform one horizontal pass

7 Perform a second horizontal pass over the same line



8 Move over 100% (no overlap) and perform one horizontal pass

9 Perform a second horizontal pass over that same line

10 Repeat process until entire cosmetic zone is treated — these are passes three and four

To perform a Clear + Brilliant® Complete treatment and give your patients the benefits of both wavelengths, treat with four passes using Original handpiece first (1440 nm) and then perform the second set of four passes, using above pass protocol, with *Perméa*® handpiece (1927 nm).

HANDPIECE SPECIFICATION AND TREATMENT LEVELS

	ORIGINAL (1440 NM)				TREATMENT SETTING	PERMÉA® (1927 NM)					
	Depth (µm)	Energy (mJ)	Coverage			Coverage		Energy (mJ)	Depth (µm)		
			4 passes	8 passes		4 passes	8 passes				
	280 µm	4 mJ	2.7%	5.4%		LOW	3.1%	6.1%	5 mJ		170 µm
	340 µm	7 mJ	4.0%	8.0%		MEDIUM	4.6%	9.2%	5 mJ		170 µm
390 µm	9 mJ	6.1%	12.3%	HIGH	6.3%	12.6%	5 mJ	170 µm			

POST-TREATMENT REMINDERS

- Redness, swelling diminish or resolve within 12-24 hours
- Cooling gels, masks, or handheld fans may help heat sensation or “sunburn” feeling
- Slightly darker skin appearance typically resolves in 4-14 days
- Dryness, sandpaper feel will gradually clear; moisturisers may help
- Skin flaking typically resolves in 4-14 days

This product is not available for purchase by the general public. Except as otherwise indicated, all product names, slogans, and other marks are trademarks of the Bausch Health group of companies. © 2024 Solta Medical, a division of Bausch Health Companies Inc. All rights reserved. Distributed in Australia by Solta Medical Australia Pty Ltd ABN 71 652 701 767. Suite 301, Level 3, 12 Help Street, Chatswood, NSW 2067, Australia. (02) 8311 0305. Distributed in New Zealand by Radiant Health Ltd NZBN 9429032273090. Pier 21 Centre, Level 3, 11 Westhaven Drive, Auckland Central 1010. 0508 35394. CLB.0027.AU.24 G24/7-101712

